

PRESTIGE HYPERBARIC

Elevate Your Recovery

Pavilion of Power

Pavilion of Power — 1.4 ATA Wheelchair-Accessible Softshell — Owner's Manual & Product Guide

The Pavilion of Power is a wheelchair-accessible soft-shell hyperbaric chamber from Prestige Hyperbaric. Specifically designed to accommodate wheelchair users and those with mobility limitations, the Pavilion allows users to enter without transferring from their wheelchair. Requires two people to install; White Glove Delivery is required if Prestige Hyperbaric handles installation.

Key Features

- **Transparent Windows:** Reinforced clear viewing windows reduce claustrophobia and brighten the space.
- **Wheelchair-Accessible Entry:** Wide, low-threshold entry designed for wheelchair users.
- **Integrated Controls:** External control panel with digital readouts for temperature, humidity, and O2 purity.
- **Power-Failure Safety:** Automatic, gradual depressurization and safe exit path.
- **Adjustable Working Pressure:** User-settable; follow your clinician's guidance for protocols.
- **Efficient Air Cooling:** Electric, maintenance-free cooling and dehumidification (no ice water/Freon).
- **Clean-Air Suite:** Anion PM2.5 purification + nano-water sterilization system.

Safety Highlights

- **WARNING — Do Not Sleep Inside the Chamber:** Never fall asleep or remain unattended while inside the hyperbaric chamber. Sessions must be conducted while the user is awake and alert at all times. In the event of discomfort, dizziness, or any unusual sensation, immediately signal an attendant or activate the pressure-release mechanism to exit safely.
- Operate only under appropriate guidance and according to the protocol recommended by a qualified professional.
- Do not use the chamber if any zipper, valve, gauge, hose, regulator, concentrator, or relief valve appears damaged.
- Keep the chamber away from heat sources, sparks, open flame, smoking materials, oils, and flammable products.
- Maintain clear communication with an attendant during use whenever possible.
- Use in a clean, dry, climate-controlled environment.
- Dual digital displays (where equipped) show O2 purity, temperature, and humidity.
- Door safety sensor prevents improper operation (hard-shell units).
- Redundant relief valves: primary + backup + emergency.
- HD phone intercom for clear communication (where equipped).
- Safe exit even during power outage after controlled depressurization.

Technical Specifications

Feature	Details
Model	Pavilion of Power

Manufacturer	Prestige Hyperbaric
Shell Type	Soft-Shell Dome (Wheelchair Accessible)
Capacity	1 person (wheelchair accessible)
Max Pressure	Up to 1.4 ATA
Oxygen Flow	10 L/min standard; 20 L/min with additional concentrator
Cooling & Dehumidification	Electric air-cooling; maintenance-free; no ice water or Freon
Purification	Anion PM2.5 purification + nano-water sterilization
Controls & Readouts	External control panel with digital readouts
Retail Price	\$13,999

Delivery & Installation Options

- **Free Curbside Delivery** — Carrier drops the crated unit at your curb. You handle offload, uncrate, move, and placement.
- **Professional Install** — We set up, calibrate, and test. Customer is responsible for moving equipment into the install room.
- **White Glove Delivery** — We handle everything: offload, uncrate, placement, on-site education, and cleanup.

Warranty

3-Year Residential Warranty / 1-Year Commercial Warranty

- **3-Year Residential Warranty (Personal Use):** Prestige Hyperbaric equipment includes a 3-year warranty when used in a personal residential setting, covering defects in materials and workmanship.
- **1-Year Commercial Warranty (Commercial / Clinical Use):** Units used in wellness centers, hyperbaric clinics, medical offices, spas, gyms, recovery studios, or any facility providing paid sessions to third parties are covered by a 1-year warranty.
- **Extended Coverage Available:** Extended warranty coverage is available for \$2,000 per additional year.
- Coverage may be refused in cases of misuse, unauthorized modification, non-professional installation where required, improper electrical setup, or use in non-climate-controlled environments. For a no-hassle experience, purchase professional installation or White Glove Delivery.

Returns

- Full refund only if canceled within 24 hours of ordering.
- After 24 hours or once shipped, orders are treated as custom and final sale unless an exception is approved.
- If a shipped return is approved, a 40% restocking fee applies and all return shipping and related costs are deducted from the refund.

Support & Contact

Website: www.PrestigeHyperbaric.com

Phone/Text: 727-591-4268

Email: prestigehyperbaric@gmail.com

Tagline: Elevate Your Recovery